

## QUICKTIP

# 2-Way Audio

## Streaming to 2.4 GHz Hearing Aids

With our 2.4 GHz Thrive platform hearing aids, the hearing aids' microphones pick up and stream the voice directly back to iPhone and iPad, allowing patients to seamlessly talk hands-free.

## Compatibility

All 2.4GHz Thrive platform hearing aids

**NOTE:** Hearing aids will require the following firmware versions:  
ITC/HS/ITE R, micro RIC 312, BTE R: 7.5.1.0 or higher  
RIC 312, RIC R, BTE 13: 6.11.1.0 or higher.

iOS 15.1 or newer


iPhone 11, iPhone 11 Pro, iPhone 11 Pro Max, and later, iPhone SE (2nd generation), as well as iPad Pro 12.9-inch (5th generation), iPad Pro 11-inch (3rd generation), iPad Air (4th generation), and iPad mini (6th generation)



## Apple Device Setup

### Native Controls

2-Way Audio is solely controlled via Apple native controls. Native controls can be accessed via a triple-click or from Control Center. 2-Way Audio is labeled “Input Options”.

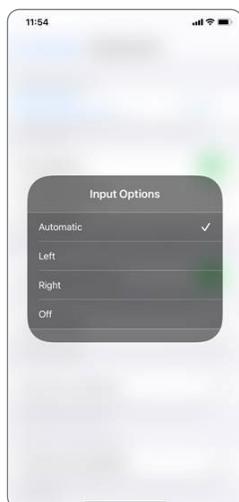

## Apple Device Setup (Continued)

### 2-Way Audio Input Options

**Automatic:** The hearing aid with the strongest connectivity to iPhone or iPad is selected and used for voice pick-up during a call.

**Left:** The left hearing aid is always used for voice pick-up during a call.

**Right:** The right hearing aid is always used for voice pick-up during a call.

**Off:** Hearing aid user is required to speak into iPhone or iPad microphone during calls. Call audio is still streamed to the hearing aids.

**NOTE:** This setting cannot be changed during an active phone call.

### Hearing Aid Control




Adding the Hearing icon to Control Center allows for direct access to various hearing aid functions.

To add the Hearing icon to Control Center:

- 1 From the Home screen, tap Settings, then tap Control Center
- 2 Tap on the green + next to Hearing to add to Control Center
- 3 The Hearing icon is now listed and appears in Control Center

Access the following functions from Control Center:

- Battery Levels
- Volume Adjustment
- Memories
- Background Sounds
- Live Listen
- Input Options

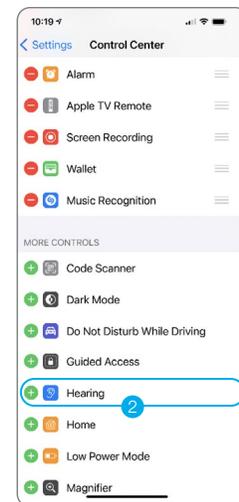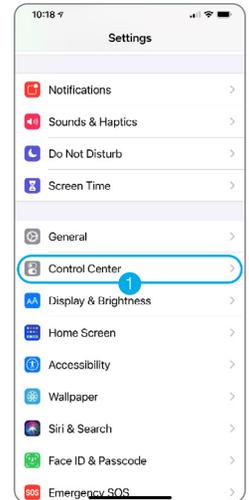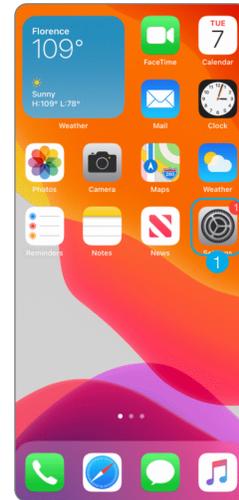

## Using 2-Way Audio

2-Way Audio can be used for phone calls as well as with other apps such as FaceTime, Teams, and Zoom.

When a phone call comes in, answer the call on iPhone or iPad.

2-Way Audio will stream based on the Input Option that was selected from Control Center.

For the best streaming experience, iPhone or iPad should be within approximately 10 feet (3 meters) of the user.

The app may look slightly different depending on your phone and/or hearing aid model.