RIC Fitting Solutions

Custom & Stock

Coupling a device with the right earmold is key to optimizing hearing aid performance. Use this reference guide to help you quickly and confidently match the perfect earmold option to each unique patient for a successful RIC fitting.

Custom RIC Solutions

Materials:

Hard Material (Digital SLS)
Soft Material (High Strength Silicone)

Colors:

Hard Material Default Color = Clear

*Soft Material Default Color = Clear

*Wide variety of additional colors available for soft material

Styles:

RIC Earmolds



RIC Hollow Canal

RIC

Helix



RIC Skeleton



RIC Occluded Helix*



RIC Canal Lock

Stock RIC Solutions

Styles:



Open Comfort Bud 5mm, 6mm, 7mm, 9mm, 11mm



Occluded Comfort Bud 5mm, 6mm, 7mm, 9mm, 11mm



Power Domes 8-10mm, 10-12mm, 12-14mm



Stock RIC mold



Stock Helix RIC Mold

How to order custom and stock RIC solutions:

Visit **eStore** or use the Earmold Order Form. You can also order through Inspire Earmold Ordering.



Fitting Reference Guide

Custom & Stock

The following chart provides an at-a-glance starting point for selecting either a custom earmold or a stock RIC fitting solution based on the patient's audiogram.

Additional custom RIC earmold styles are available. Visit our professional web site for a complete listing.

			Hearir	ng Loss			
		Mild	Mild-to- Moderate	Moderate	Moderate- to-Severe	Severe	Profound
Custom Earmolds	RIC Canal with Maximum Vent	•	•	•			Custom Fit (Embedded) Receiver 70 gain
	RIC Canal Lock with Maximum Vent	•	•	•			
	RIC Canal with Large Vent	•	•	•			
	RIC Canal Lock Medium Vent	•	•	•	•		
	RIC Canal with Small Vent	•	•	•	•	•	
	RIC Skeleton with Small Vent	•	•	•	•	•	
Stock Earmolds	Open Comfort Bud	•	•	•			
	Occluded Comfort Bud			•	•		
	Power Dome				•	•	
	RIC Stock Eamold	•	•	•	•	•	
	Stock Helix RIC	•	•	•			