


Starkey Effortless Fitting Series

Moving Beyond with Advanced Features

Judy Grobstein, AuD-FAAA, MACAud Manager of Education and Audiology
Steven Le, BSci, BAppSci, MClinaud(CCP) Consumer Support Specialist and Trainer



Hear better. Live better.

1


Starkey Effortless Fitting Series

Moving Beyond with Advanced Features

The webinar will start in..

5:00

Options for audio include computer speaker or dial-in at 02 8518 1923 / Access Code: 2633 742 4249



Hear better. Live better.

For technical questions or log-in information please contact Vincent Santana at 0413 834 718
Vincent_Santana@Starkey.com.au

2



Starkey Effortless Fitting Series

Moving Beyond with Advanced Features

Judy Grobstein Manager of Education and Audiology
Steven Le Consumer Support Specialist and Trainer



Hear better. Live better.

3

TECHNICAL DIFFICULTIES?


If not using speakers and you haven't already, please call into the call center number 02 8518 1923 and enter access code 2633 742 4249

Please be sure to keep microphones muted

If you have any technical issues, please contact Vincent Santana at vincent_santana@starkey.com.au or call him direct at 0413 834 718




4



QUESTIONS?

Please share any questions you may have in the Chat Box directed to **All Panelists**. We will do our best to answer them throughout the training.

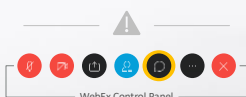

WebEx Control Panel

5


ENDORSED SESSION

This Session is endorsed for ACAud, AudA and HAASA points

You must stay logged on for the full session

AudA members must complete a 10 questions quiz with a passing score of 70% as well as your CPD Reflections and Evaluations

ACAud, HAASA and NZAS members must complete the quiz to receive full points.




6

Learner Outcomes

- 1 Participants will be able to describe how speech and music signals are individually optimized in the inspire fitting software
- 2 Participants will be able to identify the Starkey Sound noise management features and when to adjust them to enhance sound quality
- 3 Participants will be able to describe how to successfully fine-tune Starkey's frequency lowering feature

7



Starkey Effortless Fitting Series

- 1 From Basic to Brilliance
- 2 Moving Beyond with Advanced Features

8



Hearing Aid Satisfaction

HA Performance and sound	0.67
HCP Effectiveness	0.42
HA Physical Qualities	0.25
HA Maintenance	0.24
Cost (upfront and ongoing)	0.20

MarketTrak 10 / Hearing Industries Association, 2019

9




Top Drivers of Hearing Aid Performance and Sound

- Natural Sounding
Rich and Clear
- Minimize background noise
- Comfort for loud sounds
- Ability to tell direction

MarketTrak 10 / Hearing Industries Association, 2019

10




Starkey Sound™

Always on.
Always automatic.
Always effortless.


11

Effortless Hearing for the Patient


Music




Environment Management




Frequency Lowering



Fine Tuning Adjustments

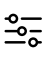



Hear better. Live better.


12




13

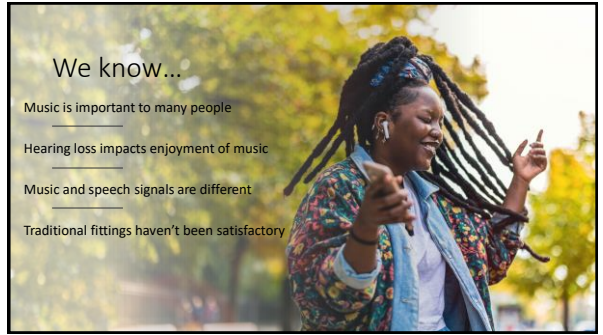


14

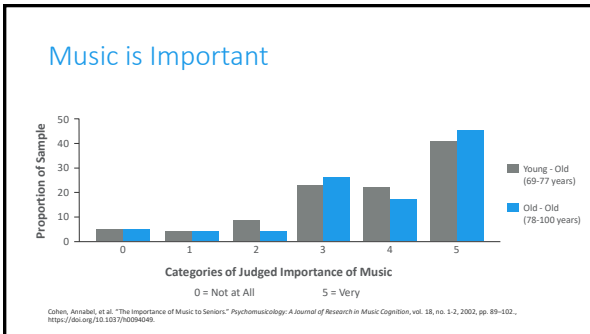

15

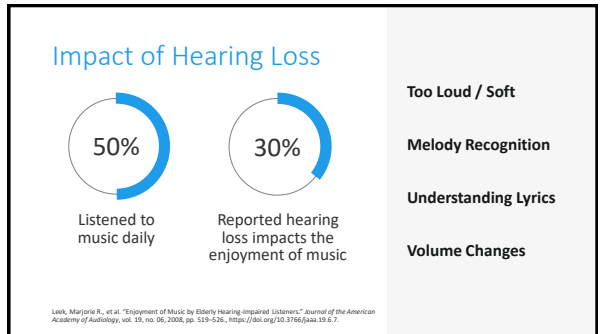

16

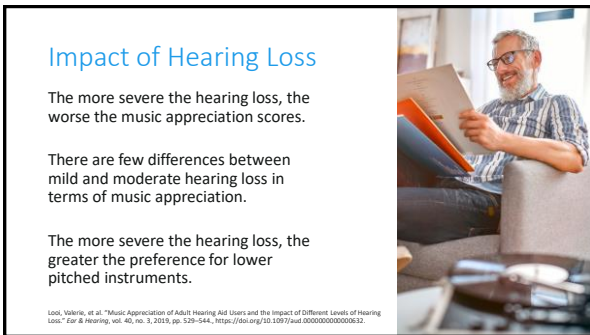

17




18

Dedicated Music memory


A memory environment that can be selected

A package of functionality including:

- A fitting rationale optimized for music
- Compression optimized for music
- Controls in Inspire optimized for music adjustment




19

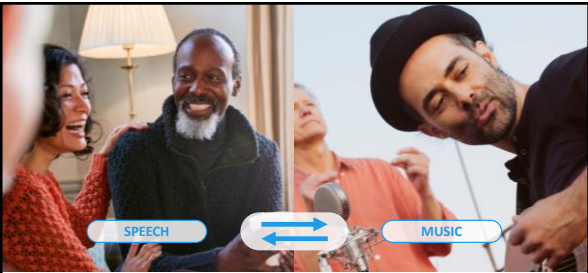

Multicore Compression Technology

20

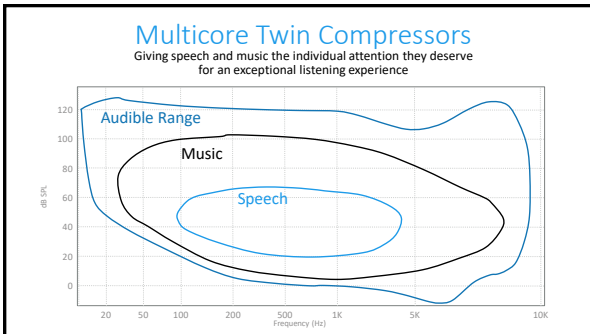


21

Dedicated Music Memory


Unique music-centric fitting prescription

Shaped gain according to hearing loss for quiet music

Flat, linear, insertion gain for loud music




22

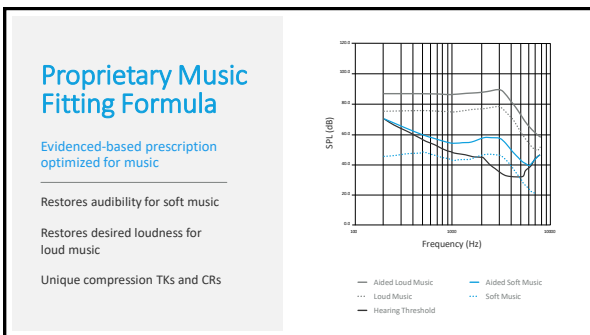


23


Unique Adjustments for Music



Omni Mic



No Noise Reduction

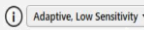

Adaptive, Low Sensitivity

Feedback Sensitivity Reduced


24

Music Memory

Fitting and evaluation of Music memory is subjective

- Individual variability of desired sound is likely much higher than for speech
- No objective measures exist for evaluating a hearing device fitting for music

Adjustments will need to be made based on patient feedback regarding music sound quality




25

Music Memory Controls

Dedicated Music memory can be programmed in multiple memory banks of the hearing device


Music-Centric controls

- To facilitate communication between patient and professional for addressing subjective music listening reports

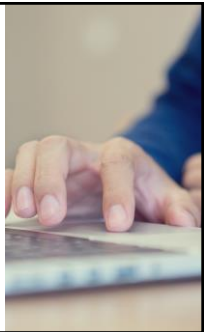


26

Dedicated Music Memory



27

Dedicated Music Memory



28



Overall Fitting Considerations

Dedicated Music Memory is for acoustic music and not speech

Stream Boost and the Thrive App can be used to control streamed music

No two patients will perceive music the same way


Individualization of programming is key for musicians and music lovers


29

Effortless Hearing for the Patient


Music




Environment Management




Frequency Lowering




Fine Tuning Adjustments

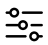





30

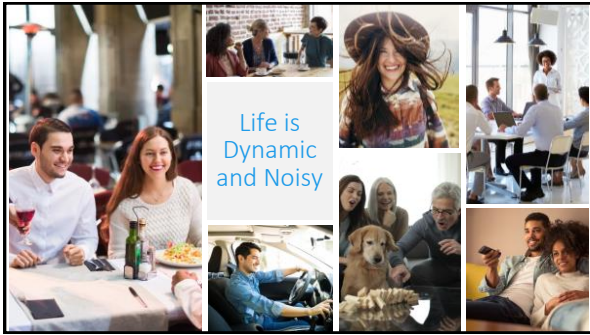


31

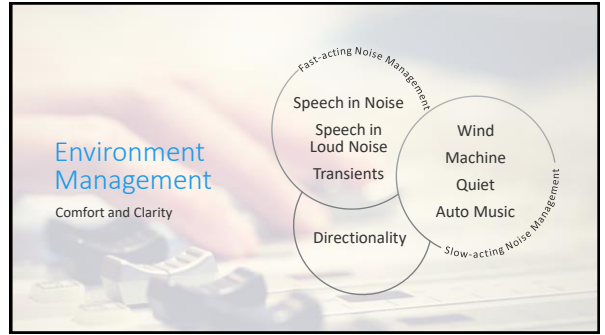


32

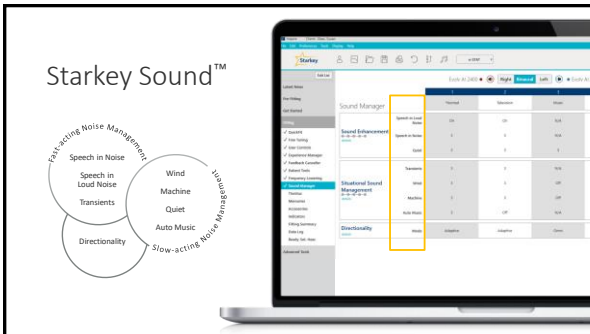


33

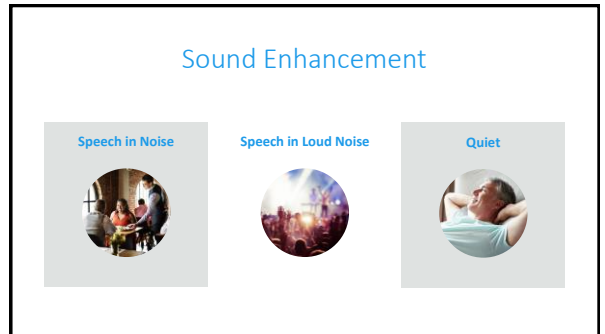

34

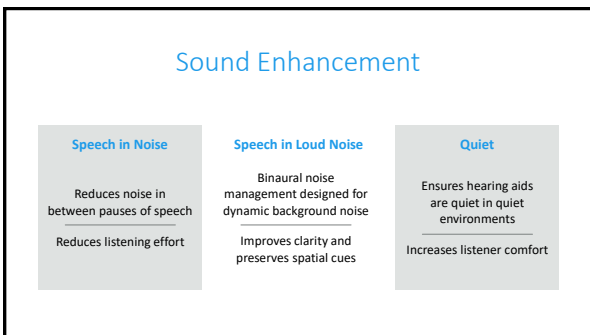

35

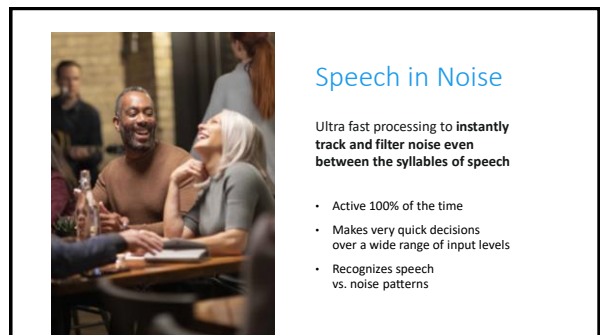


Speech in Noise

Ultra fast processing to **instantly track and filter noise even between the syllables of speech**

- Active 100% of the time
- Makes very quick decisions over a wide range of input levels
- Recognizes speech vs. noise patterns

36

Speech in Noise



OFF 1 2 3 4
 OFF 1 2 3 4
 Up to 8 dB Up to 10 dB Up to 12 dB Up to 22 dB
 1600 default 2000 default 2400 default 2400 max


System Settings with **PREMIUM TECHNOLOGY**

37

Starkey's Speech in Noise Feature

Modifies channel gain to reduce listener effort
 Significant cognitive benefits/ reduced listening effort
 Less fatigue

Listening Effort and NR

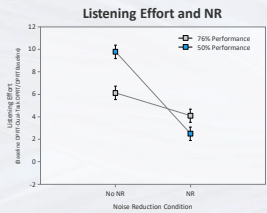

Noise Reduction Condition	75% Performance (dB)	50% Performance (dB)
No NR	~10	~6
NR	~4	~3

Starkey has the most well researched digital noise reduction algorithm in any hearing aid


Dejardin & Doherty (2014)

38

Speech in Loud Noise

Speech in Loud Noise On


- Reduces steady-state and dynamic background noise
- Reduces interfering sounds from the sides
- Preserves interaural level differences for localization



39


Quiet

Enhancing expansion with our patented fast-acting noise management, **Quiet** makes hearing aids quieter in low level environments



40

Quiet




OFF 1 2 3 4
 OFF 1 2 3 4
 Minimum Expansion Moderate Expansion Strong Expansion Maximum Expansion
 1600 default 2000 default 2400 default 2400 max

System Settings with **PREMIUM TECHNOLOGY**


41

Situational Sound Management


Transient




Wind



Machine Noise

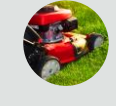

Auto Music



42

Situational Sound Management

Transient

Quickly attenuates brief loud sounds

Increases listener comfort

Wind

Reduces sound of turbulence over microphone

Increases listener comfort

Machine Noise

Reduces level of steady state noise


Increases listening comfort

Auto Music

Optimizes environmental music without compromising speech

Enhances perception of music

43



Transient Sounds

- Sensorineural hearing loss can result in a reduced dynamic range and create an abnormal growth of loudness for sound
- Recruitment makes individuals sensitive to loud sounds
- Brief impulse sounds can be uncomfortable or even jarring
 - Glass breaking
 - Dishes clanging
 - Hammer pounding
 - Door slamming

44


Transients



Off	1	2	3	4
Off	1	2	3	4
	Up to 6 dB	Up to 9 dB	Up to 12 dB	Up to 15 dB
	1600 default	2000 default	2400 default	2400 max

System Settings with PREMIUM TECHNOLOGY

45



Wind

Designed to keep patients comfortable in windy environments

46


Wind



Off	1	2	3	4
Off	1	2	3	4
	Up to 7 dB	Up to 15 dB	Up to 22 dB	Up to 30 dB
	1600 default	2000 default	2400 default	2400 max

System Settings with PREMIUM TECHNOLOGY

47



Machine Noise

Designed to deliver comfort for loud steady state noise

48

Machine Noise



OFF 1 2 3 4


1800 2000 2400 2400

5 dB 10 dB 15 dB 20 dB

System Settings with PREMIUM TECHNOLOGY

49


Auto Music



Designed to optimize the sound quality of background music while prioritizing speech and maintaining comfort


50

Gain Changes



51

Gain Changes

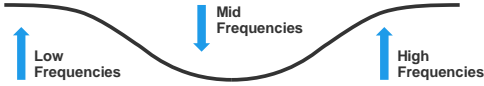

Low Frequencies

Mid Frequencies

High Frequencies

52

Auto Music




OFF 1 2 3 4

Minimum Moderate Strong Maximum

Feature change #886@igpioffsets

53

Sound Manager

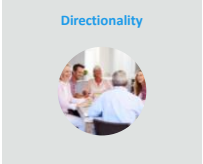

Directionality

54

Sound Manager

Directionality

Improves speech understanding in noise

Optimizes speech in noise

55

The Proof Is Conclusive: **Directionality Works**

Speech in noise intelligibility benefit 5-11 dB

The collage includes several research articles with titles such as 'Speech Recognition with In-the-Ear and Behind-the-Ear Dual Microphone Hearing Instruments', 'Unaided and aided performance with a directional', 'Real-world performance of ITE directional mic', 'Comparison of Performance of Three Directional Hearing Aids', and 'Directionality hearing in severe hearing loss'. The articles collectively demonstrate the effectiveness of directional hearing technology in various listening environments.

56

Microphone Modes

Automatic

Adaptive

Dynamic

Fixed

Directional

Omni

57

Directionality- Speech ID

This diagram illustrates how directional microphone technology is applied in a car environment. The beams are shown focusing on the front passenger and driver areas, which helps in identifying speech from those specific locations while filtering out background noise from other parts of the car or outside.

58

Examples of **SPEECH ID** Everyday

HOME

FAMILY

WORK

SOCIAL

RECREATION

59

The Industry's First and Only

Edge Mode

On-Demand Adaptive Tuning

Giving patients more control in challenging listening environments

60

Edge Mode

Puts the power of AI at the patient's fingertips to optimize hearing

Audibility

Clarity


Comfort

Provides instant adjustments to the hearing aid response, enhanced noise management, and directionality whenever the patient needs more help

61


Tap Control/User Control

2400



Edge Mode Button


2400 | 2000 | 1600
Will appear on the Home screen by default




62

Effortless Hearing for the Patient


Music




Environment Management




Frequency Lowering



Fine Tuning Adjustments





Hear better. Live better.

63


Who benefits from frequency lowering?

High frequency hearing loss

Little-to-no improvement in word recognition *with traditional approaches to high frequency amplification*


Increased distortion with appropriate high frequency gain

Reduced high frequency audibility regarding fitting variables

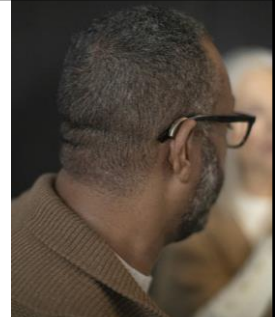

64

Precipitous High Frequency Hearing Loss

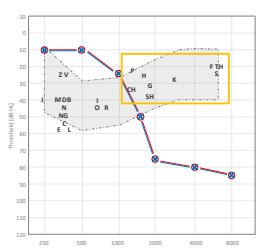


65

“s” & “z”
Over 20 linguistic uses for /s/ and /z/

“th” & “ch”
Fricatives and affricates account for about 50% of English consonants

3k Hz +
Frequencies above 3,000 Hz contribute approximately 25 percent of the audible speech cues required for recognition of spoken language



High frequency
Cues in the 3500-9000 Hz region are either important or do not create a disadvantage for many hearing aid fittings.


66

Why it's Important

Recognition of Sentences in Quiet and Noise	Spatial Unmasking
Localization	Recognition of Fricatives
Speech Clarity	Music Quality
Novel Word Learning	Ease of Listening




67




Frequency Lowering


- Intelligently identifies high-frequency speech cues
- Replicates them lower in frequency in real time to restore audibility
- Maintains a broad bandwidth
- Preserves the harmonic nature of speech




68

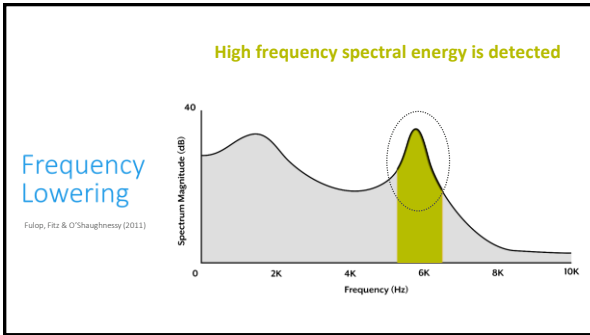


69

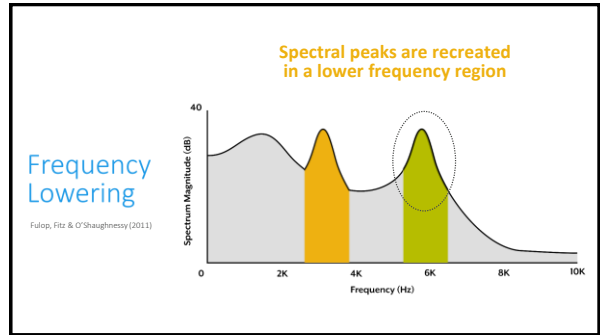


70

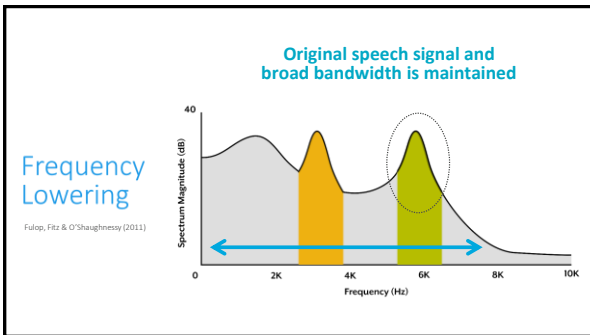

71

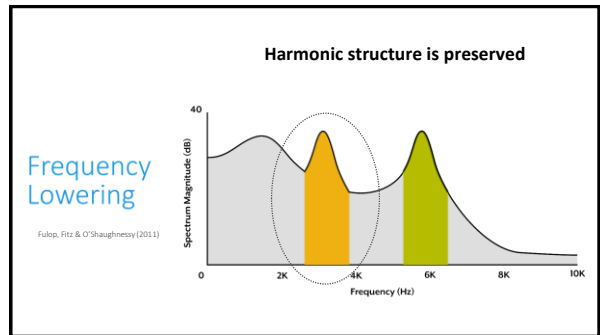


72



Professional Fitting Tools

- Smart Candidacy Criteria
- Default Logic
- Professional Controls
- Best Fit Settings



73


Smart Candidacy Criteria

All thresholds below 1000 Hz are 55 dB or better

Slope is > 25 dB per octave between 250 Hz – 4000 Hz

Any threshold between 1000 Hz – 3000 Hz is 55 dB or worse


All thresholds between 4000 Hz – 8000 Hz are 55 dB or worse

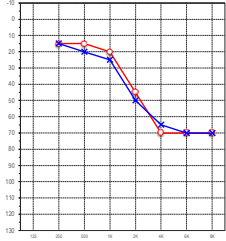

74



Frequency Lowering

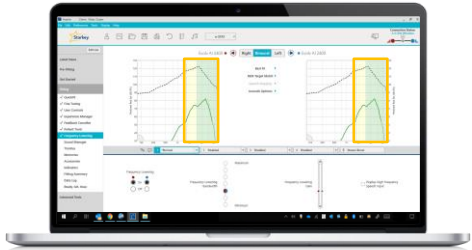


75

Source and Target Regions


- YELLOW region indicates Target Region
- GREEN region indicates Source Region

Graph Legend

- Maximum Output @ 8 SPL
- High Frequency Speech Output
- Frequency Lowering Source Region
- Frequency Lowering Target Region




76

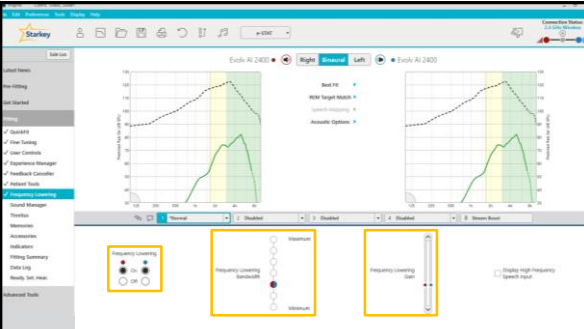

77

Professional Controls


- QuietAlert
- Fine Tuning
- User Controls
- Experience Manager
- Feedback Controller
- Patient Tools
- Frequency Lowering
- Sound Manager
- Transfer
- Accessories
- InEars
- Fitting Summary
- Data Log
- Ready, Set, Hear



78

Best Fit Settings


Frequency Lowering Bandwidth

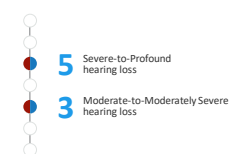

5 Severe-to-Profound hearing loss

3 Moderate-to-Moderately Severe hearing loss

Frequency Lowering Gain




3 All hearing losses

79

Frequency Lowering Bandwidth


Adjusts the **Source** and **Target** regions

7 discrete settings

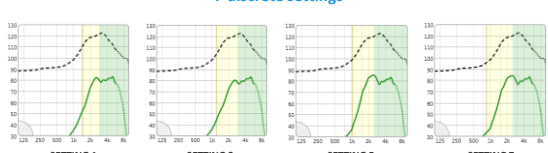

80

Frequency Lowering Gain

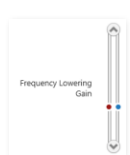

- Controls** amount of recreated signal
- Extends** from 0 dB – 10 dB
- Can be changed** in 1 dB increments

81


Sound Quality

Some patients will report altered sound quality with the feature engaged

The goal at the **time of fitting** is to focus on the **value** of the feature and **tolerance** for it

Use a **behavioral validation** tool to validate improved word recognition score

Fine tuning for sound quality should occur at the **follow-up visit**



82



83

“A listener’s **reaction** to a new processor at the time of fitting may **change** after they wear it for a few weeks. If I fine tune only in response to their **initial reaction**, I might tune away any possibility that they would get **benefit** over time. Obviously, the initial fitting needs to be **physically and acoustically comfortable**, but can also offer some **new audibility** that the wearer agrees to take home.”

Susan Scollie, Ph.D.

84

Frequency Lowering Validation

INSTRUCTIONS FOR LIVE FREE VALIDATION

1. The experimenter will read the words in LIST 1.
2. The experimenter will read the words in LIST 2.
3. The experimenter will read the words in LIST 3.

INSTRUCTIONS FOR FREQUENCY LOWERING VALIDATION

1. The experimenter will read the words in LIST 1.
2. The experimenter will read the words in LIST 2.
3. The experimenter will read the words in LIST 3.

LIST 1	SCORE	LIST 1	SCORE	LIST 1	SCORE
Key		Key		Key	
Days		Days		Days	
Plays		Plays		Plays	
Toy		Toy		Toy	
Cities		Cities		Cities	
Bricks		Bricks		Bricks	
Pig		Pig		Pig	
Cloud		Cloud		Cloud	
Ways		Ways		Ways	
Ear		Ear		Ear	
TOTAL					

LIST 1	SCORE
Key	
Days	
Plays	
Toy	
Cities	
Bricks	
Pig	
Cloud	
Ways	
Ear	
TOTAL	

85

Validation Strategy

Turn Frequency Lowering OFF

- Mask lip reading cues
- Present a sample word list
- Have patient repeat words
- Score % correct

LIST 1	SCORE
Key	
Days	
Plays	
Toy	
Cities	
Bricks	
Pig	
Cloud	
Ways	
Ear	
TOTAL	

86

Validation Strategy

Continued

If the patient scores < 80%, turn Frequency Lowering ON

Repeat the steps above

— GOAL —
To see an improvement in word recognition score with the feature ON vs OFF

LIST 1	SCORE
Key	
Days	
Plays	
Toy	
Cities	
Bricks	
Pig	
Cloud	
Ways	
Ear	
TOTAL	

87

Real World Examples

PATIENT 1

S Test Frequency Lowering OFF: 80%

S Test Frequency Lowering ON: 94%

PATIENT 2

S Test Frequency Lowering OFF: 50%

S Test Frequency Lowering ON: 70%

88

Real World Examples

PATIENT 3

S Test Frequency Lowering OFF: 30%

S Test Frequency Lowering ON: 100%

89


Frequency Lowering Studies

20 subjects

3 conditions
Off | On | Maximum

Symmetrical, mild steeply sloping to severe or profound, sensorineural hearing loss

90

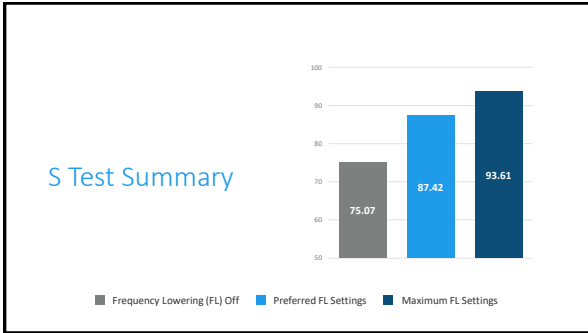

91

Win | Helps establish best bandwidth and gain settings

Win | Validation of benefit

Win | Patient counseling

92

Frequency Lowering Adjustment Guide

	PATIENT REPORT	ADJUSTMENT	CONSIDERATIONS
OWN VOICE	Own voice sounds like static or echo	Decrease Gain Decrease Bandwidth	Adjust Gain first. If reports persist, adjust Bandwidth.
	/s/ sounds too lispy	Decrease Gain	Keep in mind that a slight 'lisp' on /s/ sounds is normal
INTELLIGIBILITY	/s/ sounds too unnatural	Decrease Bandwidth	
	Difficulty detecting /s/ sounds or identifying plural sounds	Increase Gain Increase Bandwidth	Adjust Gain first. If report persists, adjust Bandwidth.
	Others voices echo	Decrease Bandwidth Decrease Gain	Adjust Bandwidth first. If report persists, adjust Gain.
SOUND QUALITY	Echo or lisp artifacts	Decrease Bandwidth	Alternatively decrease Bandwidth and Gain until report is resolved.
	Processing too noticeable	Decrease Gain	

93

Resources

94

Effortless Hearing for the Patient

Music

Environment Management

Frequency Lowering

Fine Tuning Adjustments

Starkey
Hear better. Live better.

95

Comment vs. Complaint

Let's make these more positive...

~~"Tell me the good, bad and the ugly."~~

"What do you like differently?"

"What didn't you like?"


"What surprised you most?"

96

Patient Concern

Mrs. Jordan has a severe hearing loss and has been fit with Evolv AI hearing aids and AP earmolds. She is hearing well but reports that sometimes things are "oddly unnaturally quiet" especially when she's at home.

What can you do in the Inspire® X software to help her?




97

Poll

Which feature would you adjust?

- Speech in Noise
- Quiet
- Wind
- Machine

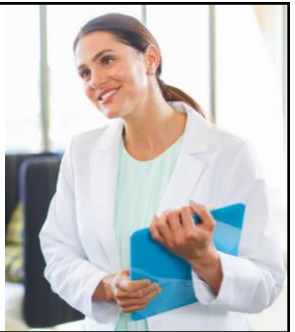


98

Solution

Adjust the Quiet Setting

Go to the Sound Manager screen.

Decrease the level of the Quiet (Expansion + Low Level Noise Reduction) setting in one or more environments to reduce the amount of noise adaptation occurring for soft sounds that fall below the WDRC kneepoint.




99

Patient Concern

Ms. Social has a lot of friends and a large family, so she is always on the go doing something.

She enjoys dining out quite often, but reports hearing conversation at the tables behind her as much as the people at her own table. She's wondering if we can fix that for her.




100

Poll

What adjustment would you make to help your patient with her report of hearing people at another table better than those she's sitting with?

- Increase her MPO
- Change Directionality in her Restaurant Memory to Omni
- Change Directionality in her Restaurant Memory to Directional
- Ask her to have her friends at the table speak louder




101


Solution

Change mic setting to Directional

Adjust gain/compression




102



Patient Concern

Dr. Landon is taking dance classes with her husband and wants to know if there is a way to manage music so that it doesn't overpower the conversation with her husband or the instructor.

What can you do in the Inspire® X software to address this issue?




103

Solution

Adjust the Auto Music Feature

- Access Auto Music from the Sound Manager page
- Click on the Details under Situational Sound Management
- Decrease the Auto Music setting so the music is not as robust.



Auto Music

When music is detected in the environment, the hearing aids will automatically adjust their settings to provide an optimal music listening experience. When speech is detected, the hearing aid settings will change back to those optimized for speech listening.


The Music Strength control adjusts the amount of gain adaptation based on music listening settings that occur when music is detected.

104

Patient Concern


Ms Smith reports the sound in her hearing aids will sometimes fade in and out....this is very bothersome to her."

How can we address this issue in Inspire?



105

Default Sound Manager Settings

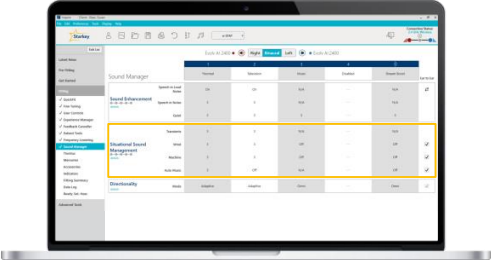


106

Patient Concern

Ms. Jones is wearing Evolv AI 2400 RIC R hearing aids, has a precipitous high frequency hearing loss and reports that when she is talking to her friends, the /s/ sounds are a bit lispy.

What can be done to help her with this?




107

"/s/ sounds are too lispy"

Which frequency lowering adjustment would you make?

- Increase FL Gain
- Decrease FL Gain
- Increase FL Bandwidth
- Turn off frequency lowering



108

“/s/ sounds are too lispy”

109

“Speech sounds unnatural”

110

Have patience and be creative

There is no one right answer

Be flexible and willing to try a variety of solutions

111

Expert Assistant

112

Questions

Judy Grobstein, AuD-FAAA, MACAud
Judy_Grobstein@Starkey.com

113

We're Here to Support You

Professional
Audiology Support

1800 024 985


Contact your
Product Specialist

 Vincent Santana
NSW/SA
  Connie Lu
VIC/TAS
  Phil Nelson
QLD/NT
  Georgina Allam
WA/ACT


114



115

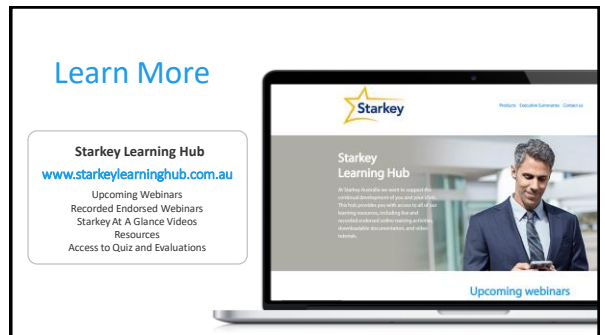

116

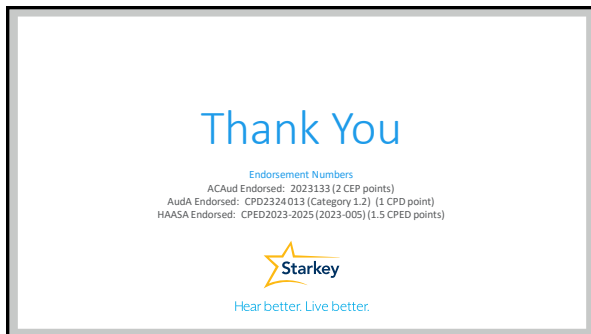

117