

QUICKTIP

My Starkey

Android Version

Smart Assistant
Tap to use voice commands to change programs, set reminders, ask questions on hearing aid topics, and more.

Battery Status
L R

Current Program
Personal
Your everyday hearing setting

Program Options
Access quick details on the current program, make sound adjustments, and more.

Edge Mode +
Tap to get the best sound instantly for the current environment.

My Hearing
Manage hearing aid features and settings.

Programs
Access programs or create a custom program.

Binaural Volume Control

Accessories
Manage hearing aid accessories.

Monaural Volume Control

Return to Home
Home Health Learn More

Health
View progress and insights on hearing and physical activity (steps, exercise, and more).

Learn
Access helpful how-to's, tips, and videos on hearing aid topics.

Downloading My Starkey

You must have a Google Play Store account to download My Starkey. A Google Play Store account can be created by going to **Settings>Accounts**. Choose an existing account or select **Add Account** to create a new account.

- 1 Once an account is set up, open the Play Store.
- 2 Search for **My Starkey**.
- 3 Select **Install**.



My Starkey

My Starkey Pairing and Initial Setup

My Starkey will search for and pair to the hearing aids. Pairing should be completed after the hearing aids have been programmed in the Pro Fit software. Ensure that Pro Fit is closed prior to pairing with any Android device.

- 1 Tap on **Get Started** to begin the app set-up process.
- 2 Review the Terms of use. Tap **Accept** to continue.
- 3 Sign in, create an account with an e-mail and password, or select **Skip for now** to postpone account creation. Cloud connectivity allows for TeleHear, Fall Alert and for app/hearing aid information to be backed up and restored.
- 4 Users are asked to allow My Starkey to collect and store Hearing aid data, Activity tracking, and Analytics. Select **Allow** or **Not Now**.
- 5 Select **Continue** to allow My Starkey access to the user's location and to share notifications.
- 6 Users are asked to allow My Starkey to send the user notifications. Select **Allow** or **Don't Allow**.
- 7 Ensure the hearing aids are charged and turned on. Select **Continue**.
- 8 A prompt will inform the user that Nearby Devices Permission is required. Tap **OK**, then select **Allow**.
- 9 When prompted, select **Pair**, then tap **Agree** to allow My Starkey to access the hearing aids.
- 10 My Starkey will connect to the hearing aids. Select **Continue**.

You're All Set. Select **Got it** to enter My Starkey.

Features may vary by country.

The app may look slightly different based on your phone and/or hearing aid model.
Android and Play Store are trademarks of Google, LLC.
Starkey, Starkey logo, Pro Fit, Edge Mode and TeleHear are trademarks of Starkey Laboratories, Inc.
©2024 Starkey Laboratories, Inc. All Rights Reserved. 11/24 FLYR4084-02-EN-ST

